



PathoSense

2025

CONFIDENTIAL

The content of this document is confidential and intended for the training of employees of official PathoSense Partnerlabs. It is strictly forbidden to share any part of this document with any third party, without a written consent of PathoSense.



Wet lab flow diagnostics PathoSense

WET LAB I - Enrichment

Viral enrichment via Benzonase-Nuclease

Timing ~ 45 min



PathoSense highly recommends to perform the enrichment step **daily**, so that samples are stabilised as soon as possible after arriving in the lab.

Host ≠ Cat/Dog

Reagents

- IC + HMB, provided by PathoSense, aliquoted by Partner Lab
- 10 nM EDTA, provided by PathoSense
- Benzonaze® Nuclease, Purity > 90%, 70746 Millipore = enzyme, so keep on -20°C!

REAGENT	QUANTITY per sample	HANDLING CONDITIONS
IC+HMB	1 aliquot	on ice
Benzonase nuclease	3.5 µL	-20 °C (on cool rack)
10 nM EDTA	12.5 µL	on ice

Materials

- 1,5 mL Eppendorf tube
- 1,2 µm filter
- PathoSense Sample



- All surfaces should be cleaned with Hibiscrub, 70% Ethanol, and 10% Bleach in this order of cleaning. Prevent contamination.
- Aerosol-barrier tips should be used throughout the entire procedure.
- Perform all steps in the dedicated sample preparation area.

Preparations

- Pre-heat the **heat block at 37°C**
- Thaw a 1.5 mL Eppendorf tube with **an aliquot FeCVII internal control (IC) + 20x Homemade Buffer (HMB)** on ice **just before adding the sample** (because the IC contains a virus)

Protocol

1. **Label a 1.5 mL Eppendorf tube** for each sample
2. Connect a **filter to the syringe** containing the sample
3. **Flush the sample through the filter** in the corresponding labeled Eppendorf tube
4. Take a **1.5 mL Eppendorf tube with an aliquot FeCVII internal control (IC) + 20x Homemade Buffer (HMB)** for each sample

- Transfer **200 µL of the sample** to the Eppendorf with the aliquot FeCVII internal control (IC) and 20x homemade buffer
- Add **3.5 µL Benzonase -Nuclease** to each tube
- Gently **mix by inverting tube 3 times** (DO NOT VORTEX) and quick spin to collect all droplets
- Incubate the tubes for **30 minutes** at **37°C**
- Add **12.5 µL 10nM EDTA** to each tube to stop enzymatic reaction
- Gently **mix by inverting tube 3 times** (DO NOT VORTEX) and quick spin to collect all droplets

Safe stopping point

If the samples cannot be processed immediately, Benzonase-treated samples can be stored at -70°C for 1 month

Keep Benzonase-treated samples on ice and proceed with

WET LAB II EXTRACTION

Host = Cat/Dog

Reagents

- HMB, provided by PathoSense, aliquoted by Partner Lab
- 10 nM EDTA, provided by PathoSense
- Benzonaze® Nuclease, Purity > 90%, 70746 Millipore = enzyme, so keep on -20°C!

REAGENT	QUANTITY per sample	HANDLING CONDITIONS
HMB	12.5 µL	on ice
Benzonase nuclease	3.5 µL	-20 °C (on cool rack)
10 nM EDTA	12.5 µL	on ice

Materials

- 1,5 mL Eppendorf tube (2)
- 1,2 µm filter
- PathoSense Sample



- All surfaces should be cleaned with Hibiscrub, 70% Ethanol, and 10% Bleach in this order of cleaning. Prevent contamination.
- Aerosol-barrier tips should be used throughout the entire procedure.
- Perform all steps in the dedicated sample preparation area.

Preparations

- Pre-heat the **heat block at 37°C**

Protocol

1. **Label a 1.5 mL Eppendorf tube** for each sample
2. Connect a **filter to the syringe** containing the sample
3. **Flush the sample through the filter** in the corresponding labeled Eppendorf tube
4. Take a **1.5 mL Eppendorf tube** and transfer **234 µL of the sample**
5. Add **12.5µL 20x Homemade Buffer** (HMB) for each sample to the Eppendorf tube
6. Add **3.5 µL Benzonase -Nuclease** to each tube
7. Gently **mix by inverting tube 3 times** (DO NOT VORTEX) and quick spin to collect all droplets
8. Incubate the tubes for **30 minutes** at **37°C**
9. Add **12.5 µL 10nM EDTA** to each tube to stop enzymatic reaction
10. Gently **mix by inverting tube 3 times** (DO NOT VORTEX) and quick spin to collect all droplets

Safe stopping point

If the samples cannot be processed immediately, Benzonase-treated samples can be stored at -70°C for 1 month

Keep Benzonase-treated samples on ice and proceed with

WET LAB II - EXTRACTION